

HISTOLOGICS



THE FUTURE IS ... COMPASSIONATE COLPOSCOPY® EXOCERVICAL BIOPSY AND ENDOCERVICAL CURETTAGE

Choose the Patient-Friendly "Gentle" Approach to Biopsy and ECC

High Quality Colposcopic Biopsy and Curettage with the Patient Care Experience in Mind!

- **ABUNDANT BROAD SAMPLE**
- **AFFORDABLE**
- **HIGHLY EFFICIENT TISSUE TRAPPING**
- **DIAGNOSTIC TRANSEPIHELIAL SAMPLES**

EXOCERVICAL BIOPSY DEVICES

SoftBiopsy®

The Kylon® fabric both removes and traps abundant transepithelial (full thickness) samples.
EASY, RAPID, GENTLE, ABUNDANT SAMPLE from a broad area of the exocervix.



Spirabrush CX®

Stiff bristles in spiral array remove abundant transepithelial (full thickness) samples.
EASY, RAPID, GENTLE, ABUNDANT SAMPLE from a broad area of the exocervix.



EXOCERVICAL BIOPSY METHOD FOR BOTH DEVICES

1. Visualize the cervix under colposcopy, choose the target area for biopsy
2. Press on the cervical lesion with moderate pressure to create friction.
3. Six half wrist rotations like "key turning" in clockwise, then counter-clockwise.

ENDOCERVICAL CURETTE DEVICE

Soft ECC® & Soft ECC-S®

The Kylon® fabric both removes and traps abundant transepithelial (full thickness) samples.
RAPID, GENTLE, ABUNDANT SAMPLE from the endocervical canal. No retrieval needed - FAST & EASY.



ENDOCERVICAL CURETTAGE METHOD FOR BOTH DEVICES

1. Clinical decision to perform endocervical curettage for colposcopy or bleeding problem.
2. Gently insert fabric tip in to canal with moderate pressure.
3. Six half wrist rotations like "key turning" in clockwise, then counter-clockwise.

YES! There is a high quality patient-friendly frictional alternative to the conventional stainless steel "cutting" punch biopsy forceps and sharp scraping curettes!



ORDER NOW (Order Form on back)