


Pictorial Guide: Instruction for Use :SoftBiopsy® : Histologics LLC 2012


Question 4 - Figure 1

Place the bottom tip of the SoftBiopsy® in the center of your palm.

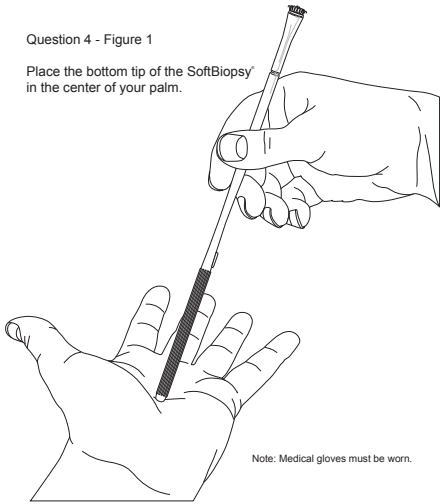


Question 4 - Figure 2

Close your hand around the SoftBiopsy®, with your fingers along the ridges, and your forefinger on the tactile notch.

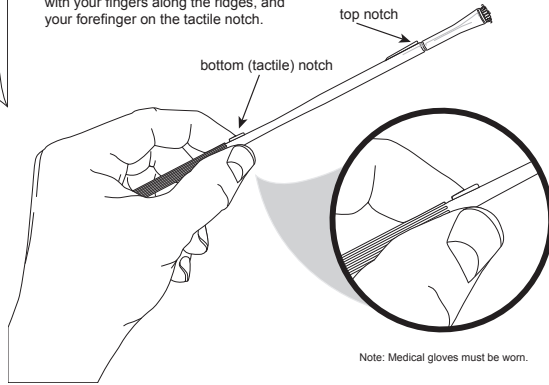


Question 4 - Figure 3
Single Hand Rotation Technique

Fingers will rotate SoftBiopsy® device. Notches serve to keep track of rotations.

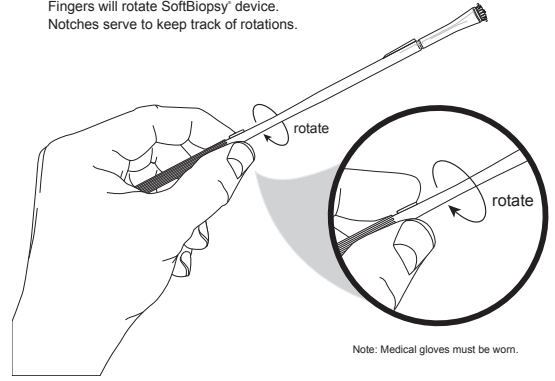


Question 4 - Figure 3.5
Two Hand Rotation


The handle is rotated with one hand while the pincher fingers are placed at the tapered waist at the mid shaft. This is smooth and permits guided rotation or 180 degree agitation motions.

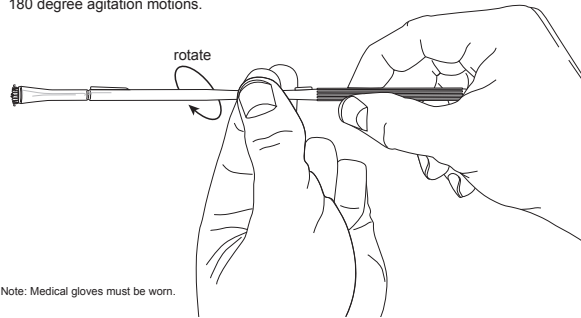


Question 4 - Figure 4

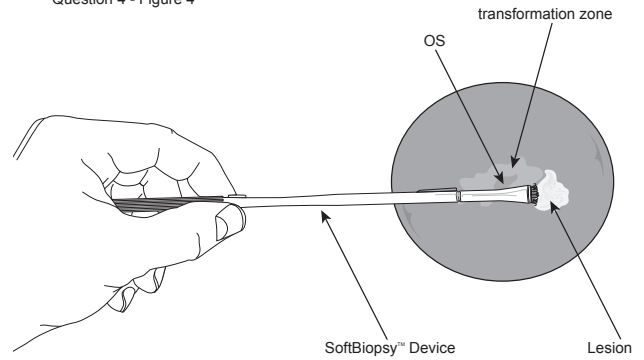


Question 4 - Figure 4.2

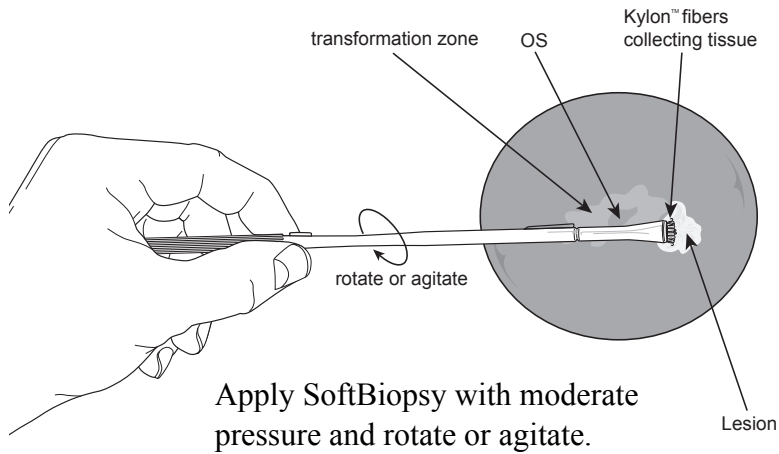

Question 4 - Figure 6

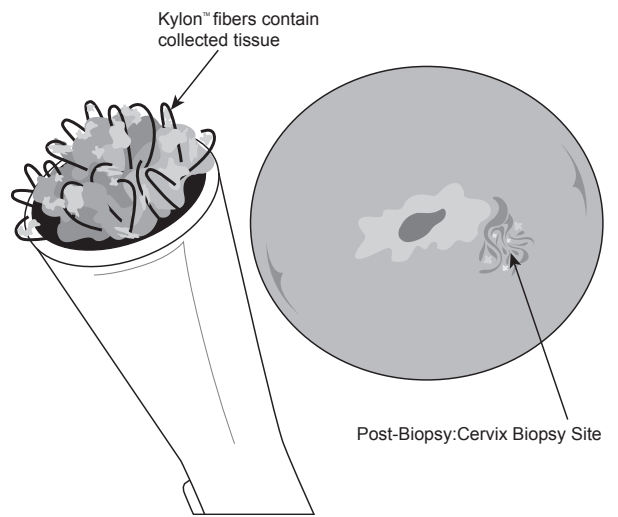

Question 4 - Figure 7

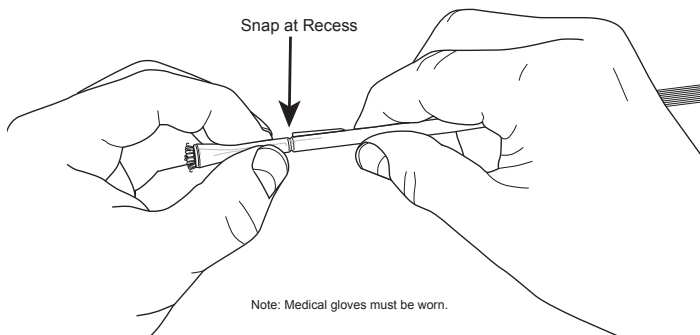

Question 4 - Figure 8

